

1. Linkor Programmation

- Settings

Turn on the device by pressing the switch on the back: 5 windows will appear successively:

Select **language, unit, region, time**..... **page 10 of the User Manual**


Log in to account : **ID: immunec - Password: linkor (lowercase)**

Connect to the internet:  **Wi-Fi or Ethernet cable**..... **page 11 of the User Manual**


An automatic update will start after a few minutes: **"Accept"**.

Once the update is complete, the device will restart. The Linkor is now ready to use.

To access the settings, click on the icon 

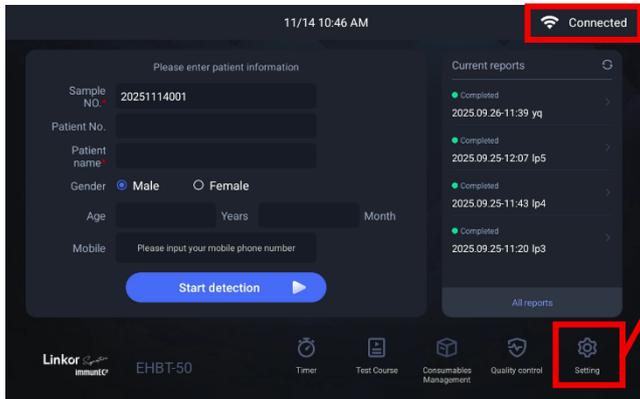

Scroll through the touchscreen to access all settings.

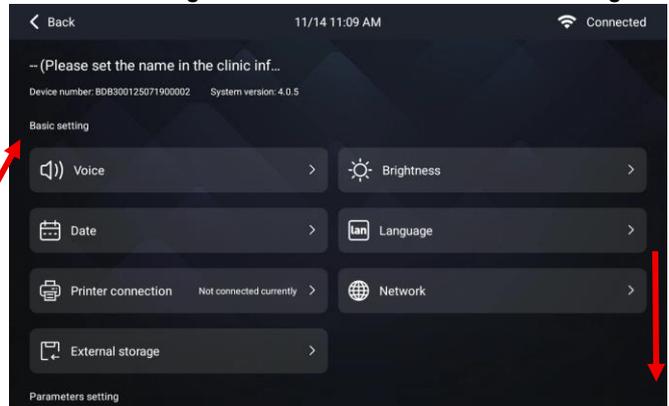

2. Print test reports

- On thermal paper (included)..... **page 10 of the User Manual**

Installing the printing paper:

Open the thermal printer housing located on the right side of the Linkor.

Pull the lid up. Simply place the roll of printing paper into the paper tray.

Make sure that the drop-down part of the printing paper is facing downwards.

Allow the printing paper to protrude slightly from the tray and close the lid.

Select the desired test report and click on the print button at the top right.



- On A4 paper..... **page 10 of the User Manual**

In the **[Settings]** menu, click on **"Print connection"**

Click on the **"Connect Printer"** button to access the **[Printer Connection]** page.

[Printing Device] means the printer that can be detected via Wi-Fi or on your local network.



3. Export test reports to PDF

- **To USB** : Plug a USB stick into the back of the machine (USB port)..... **page 14 of the User Manual**
Select the desired test report and click on the export button at the top right.

Maintaining a USB device in the rear port is advised to facilitate data recovery

- **To Smartphone or PC** , phone, Linkor are imperatively connected to the same network.

Click [Settings]    Report Viewing Address  **The IP address** of the Linkor is displayed. (Ex: http:192.168.1.104:8000)
Enter this IP address into the **browser of your phone or PC.**  **ENTRY**

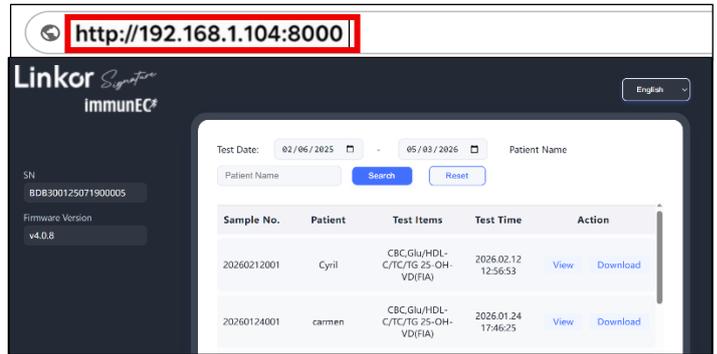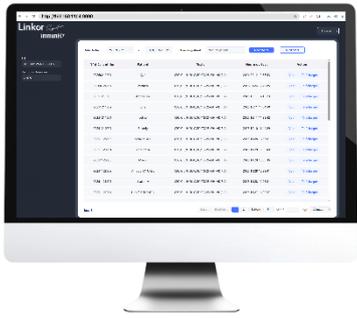

Test Report Access Web Page

4. Reagent Kits


Each reagent box contains an NFC card with information about the batch and calibration curve.

Before using a new reagent kit, record the batch number of each of the 3 reagents.

See video: Description of Reagents (immunec.com site: tab: Support)

 Remember to equip yourself with: **cotton swabs, bandages and alcohol wipes.**

NB: Before loading the reagents into the cart, please remove the collector tape from the hematological rail.

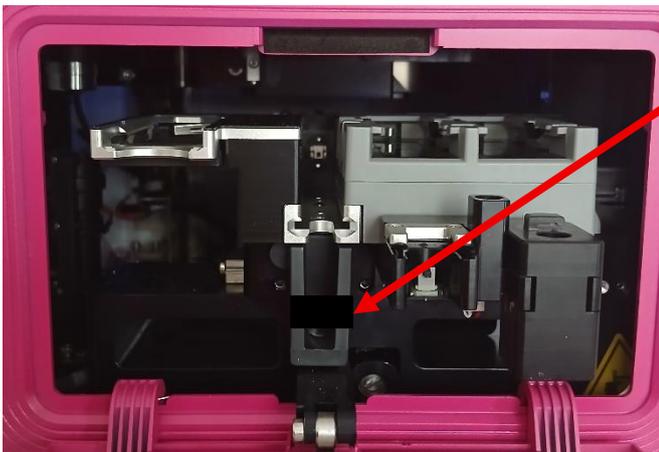

This adhesive tape ensures the maintenance and stability of the hematological collector only during

6. Troubleshootingcf. page 22 of the User Manual

Download the **support request form** from the **immunec.com** site: Warranty **tab** of the menu.

